

CRYOcheck™ **IVD** C & S DILUENT

Intended Use

CRYOcheck C & S Diluent is a buffer solution intended for use as a diluent for reagents and patient samples in coagulation tests.

Reagents

C & S Diluent is an optimized buffered solution containing sodium azide (<0.1%) as a preservative.



Sodium azide may react with lead and copper plumbing to form highly explosive metal compounds. Ensure proper disposal of reagent according to federal, state, and local regulations.

Storage, Preparation and Handling

When stored at 2 to 8 °C, C & S Diluent is stable to the end of the month indicated on the product packaging.

Allow refrigerated diluent to acclimate to room temperature (18 to 25 °C) for 30 minutes prior to use.

C & S Diluent may be used for up to three days after opening when maintained on a coagulation analyzer at 15 to 20 °C.










Availability

Product	Catalog #	Format
CRYOcheck C & S Diluent	CSD	6 vials x 15.0 mL

Instruments

Each lab should prepare the local instrument in accordance with the manufacturer's instructions for use.

Symbols Used

	In vitro diagnostic medical device		Warning
	Batch code		Manufacturer
	Catalogue number		Authorized representative in the European Community / European Union
	Use by date	Rx ONLY	For prescription use only
	Temperature Limit		Consult electronic instructions for use



European Authorized Representative (Regulatory affairs only)
Emergo Europe— Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands



Precision BioLogic Inc.
140 Eileen Stubbs Avenue | Dartmouth, Nova Scotia | B3B 0A9 | Canada

Tel: 1.800.267.2796 / +1.902.468.6422

Fax: 1.800.267.0796 / +1.902.468.6421

www.precisionbiologic.com